


ADAPTOR FOR URINAL BULBOUS

Biodegradable Moulded Pulp

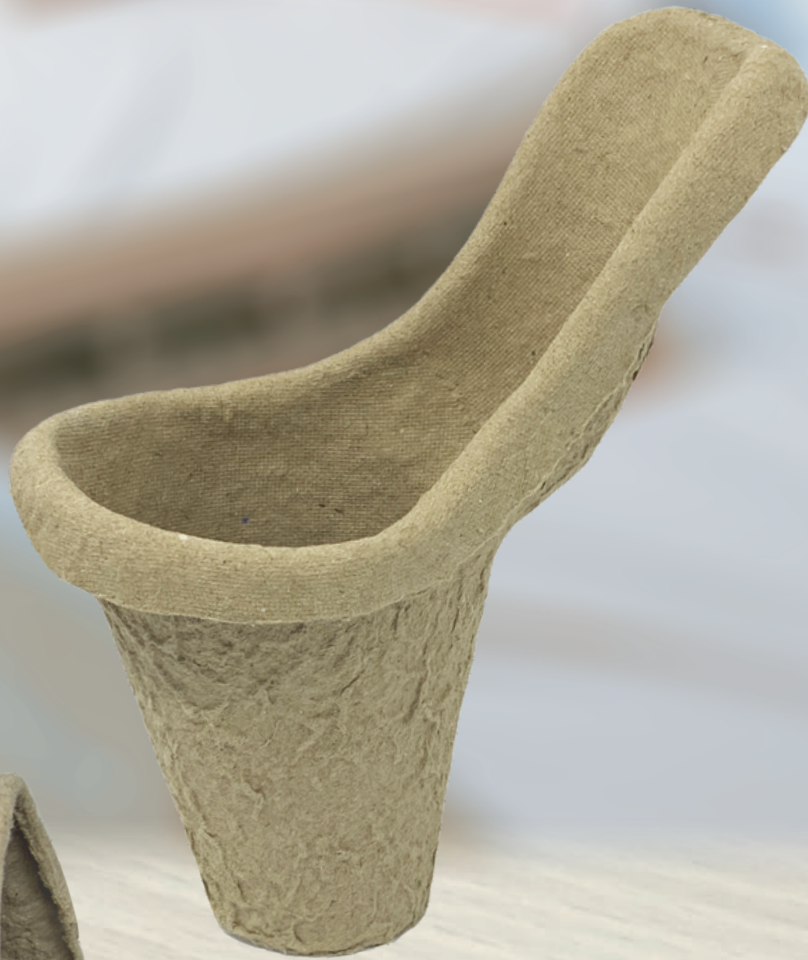

Designed for Single Use so as to Reduce the Risk of Infection



The biodegradable Adaptor for Urinal Bulbous is easy to use and environmentally friendly. It is used as a female adaptor fitted on the male urinal bulbous, allowing females to use the urinal bulbous to promote self-toileting.

This is design for single use so as to reduce the risk of infection. This eco-friendly adaptor for urinal bulbous is made from recycling biodegradable waste paper and is macerator friendly.

Suitable for Use



Hospitals



Nursing Homes



Personal Use




Medical Clinics




Disability Homes


User Guide

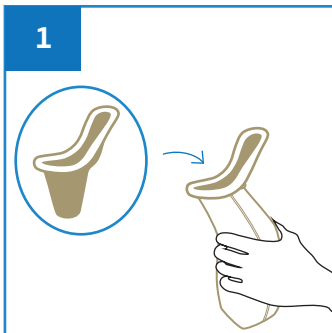


1
Insert Adaptors into Male Urine Bulbous.

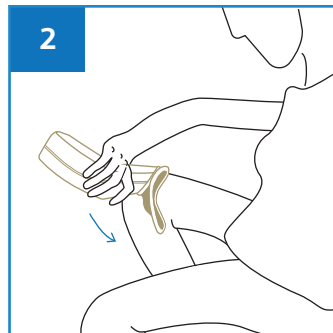

2
Place the Female Urinal Bulbous between the legs and cover completely for urination.

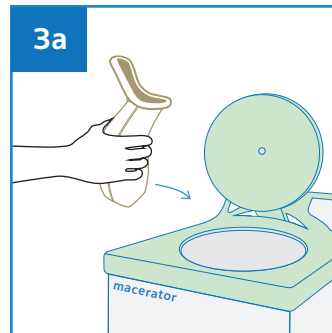

3a
Dispose the Female Urinal Bulbous with the urine into macerator.

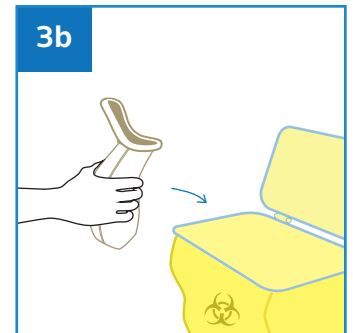

3b
Empty the urine into toilet bowl before disposing the Female Urinal Bulbous to appropriate waste bin.



Stackable



Macerator Friendly



Eco-Friendly



Recycling

PRODUCT INFORMATION

Adaptor for Urinal Bulbous (Female Urinal)

DIMENSION

120 x 90 x 100 mm

PACKAGING

100 pcs per carton

PRODUCT CODE

AUB1000



Manufactured in accordance with PAS29:1999
Disposable Pulp Products For Use in Healthcare

Rev0(0222)EN