

## Medication preparation range

# Tookan<sup>®</sup> pill crusher

The Tookan<sup>®</sup> pill crusher enables to reduce drugs to **powder**. As it is equipped with an integrated handle which allows to crush all medicines without any efforts, it is suitable for anybody.

Thanks to the storage of bags in the Tookan<sup>®</sup>, **no loss neither contamination** are possible with other medicines. Hygiene is therefore completely respected.



Hygienic



Ergonomic

## Advantages

- ▶ Easy use
- ▶ Resistant to disinfecting products of hospital environments
- ▶ Crushing without any effort
- ▶ Bags are stored in the crusher

## Medication preparation range

# Tookan<sup>®</sup> pill crusher

### Technical features

#### Tookan<sup>®</sup> pill crusher

- ▶ Steel base with epoxy coating
- ▶ Aluminium handle
- ▶ Anodised aluminium crushing part
- ▶ Stainless steel axis

#### Bags for pill crusher

- ▶ Transparent low density polyethylene
- ▶ Thickness: 180/200 µm
- ▶ Environment-friendly



Title	Dimensions (in mm)	Reference
<b>Manual pill crusher Tookan<sup>®</sup></b>	L. 265 x D. 75 x H. 90	NA00017
<b>Batch of 1000 bags for Tookan<sup>®</sup></b>	L. 50 x H. 100	NA00011.01

### Operating instructions



Take a bag in the storage compartment.



Insert medicines into the bag and fold it up.



Put the drugs into the polyethylene bag and place the bag into the crusher.



Press smoothly on the handle with fingertips without releasing the bag.



You can take the drugs which are now reduced to powder.