

PLASTIC SLIPPER PAN & BEDPAN

with Fitted Lid and Integrated handle



Available
in **two**
Forms



Intergrated
Handle



Intergrated
Lid



Up to
2000ml



Reusable
After Washing

PLASTIC SLIPPER PAN & BEDPAN




It is common for people to have difficulty in bowel incontinence while recovering from an extreme operation. As such, plastic slipper pan or bedpan is used to assist people in their sanitation needs.

This is a reusable thermoplastic pan with integrated handle and fitted lid to assist users with limited hip mobility to go about their toileting in bed when it is convenient. Suitable for both bladder and bowel incontinence.

Ergonomically Designed

The design of the slipper pan and bedpan is hygienically safe and easy to use. It is made of thermoplastic and is ideal for users confined to the bed or inconvenient to go to the toilet.

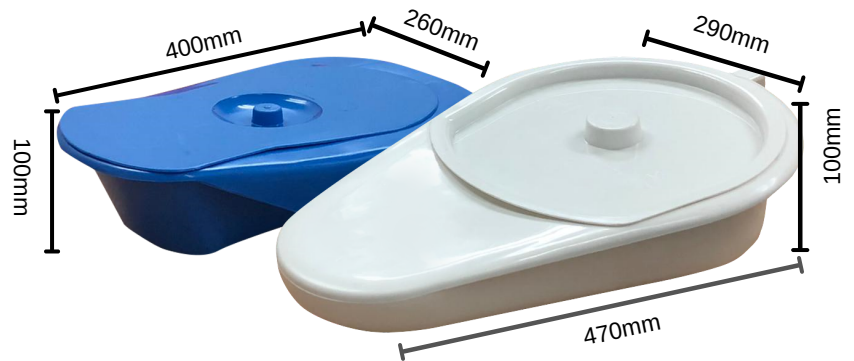

Easy to Clean

The smooth internal surface enables easy cleaning and drying after use. Can be washed with up to 135 degrees Celsius.

Tightly Fitted Lid Cover


When the fitted lid is on it will prevent any unpleasant smell and sight coming from the slipper pan.




Integrated Handle

The integrated handle allows for an easy grip to slip the slipper pan or bedpan under the user. The integrated handle allows for ease of emptying contents into the toilet bowl after each use.


GUIDE TO USAGE

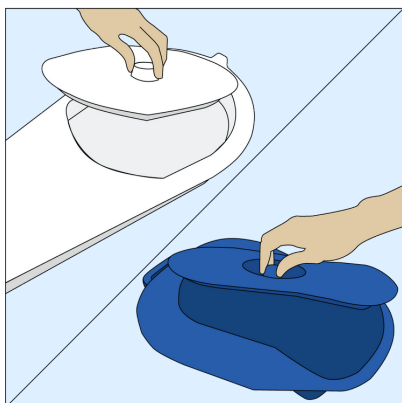


1. Open the lid of the slipper pan before using.

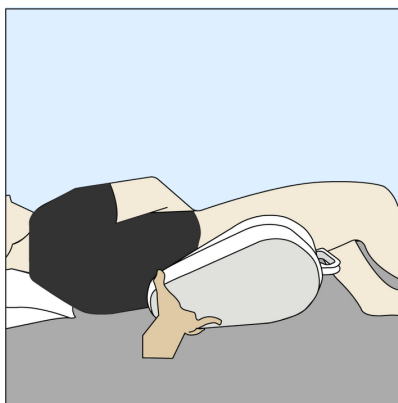


2. Turn the user to the side, gently press the slipper pan or bedpan against users bottom.

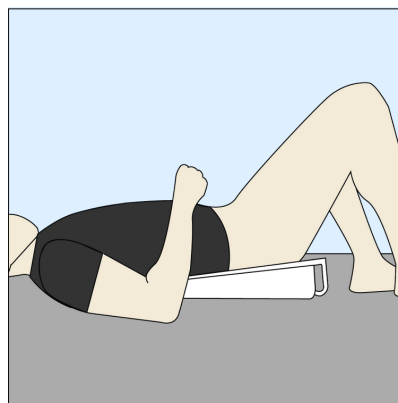


3. Turn the user back. The user is ready for toileting. For privacy, leave the person alone when it is safe to do so.

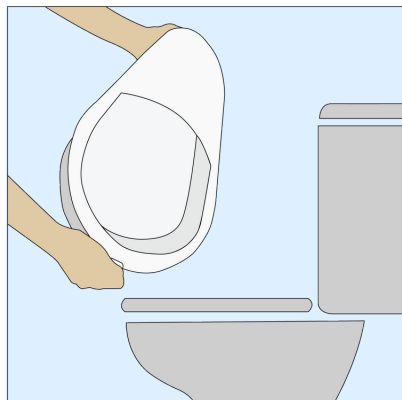

4. Empty the pan into the toilet bowl and flush after use.

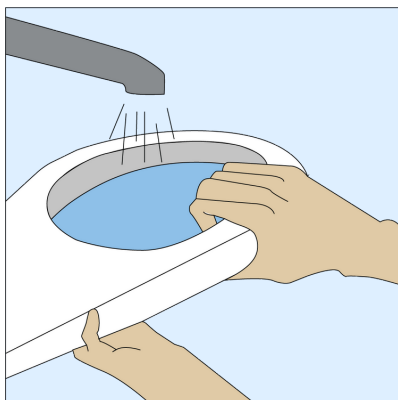

5. Wash the slipper pan or bedpan with disinfectant and rinse thoroughly with water.

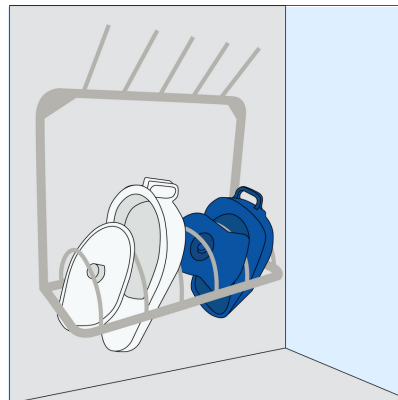

6. Keep lid open. Dry the slipper pan and bedpan on a rack. Ensure plastic slipper pan or bedpan is dry before subsequent use.

***This product information does not provide professional medical advices**

The user guide of this product is provided for general information purposed only. it is not intended to replace consultation with a healthcare professional or to provide medical advice, diagnosis or treatment. Always seek the advice of a physician or other qualified health care provider with any medical or health questions. Never disregard professional medical advice or delay in seeking it because of something you have read about this product

PRODUCT INFORMATION

CAPACITY


PRODUCT CODE



Plastic Slipper Pan with Lid and Integrated Handle

2000ml

PSP2000W

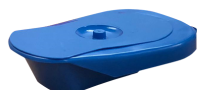

Plastic Bedpan with Lid and Integrated Handle

2000ml

PBP2000B

*Disclaimer

The user guide on this product is provided "as is" without warranties or conditions of any kind either expressed or implied. To the fullest extent possible under applicable law, NEU Industries Pte Ltd disclaims all warranties and conditions, expressed or implied, including but not limited to, implied warranties or conditions of merchantability and fitness for a particular purpose, non-infringement or other violation of rights. NEU Industries Pte Ltd does not warrant or make any other representations regarding the use, accuracy, timelines, applicability, performance, security, availability or reliability of this product, or the results from the use of this product.



132 Joo Seng Road #04-01 Singapore 368358

Tel: 6817 8969 Fax: 6636 9813

sales@apexpulp.com

www.apexpulp.com