


OPERATING INSTRUCTIONS

PULPMATIC ULTIMA


MicrobeSafe[®]
Antimicrobial Surface

HANDS FREE OPERATION

- 1 Open Lid**
Via optical foot sensor

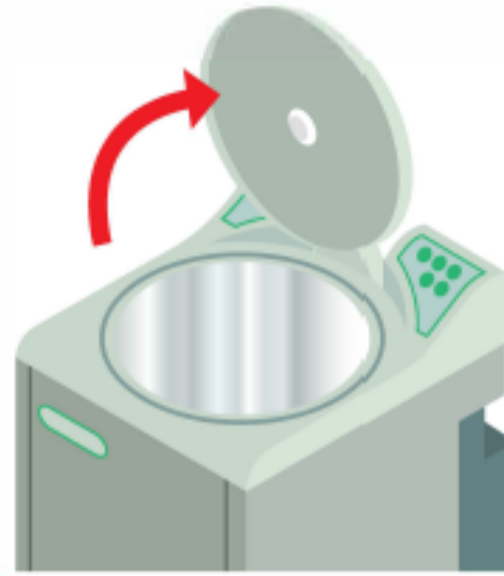


MACHINE READY

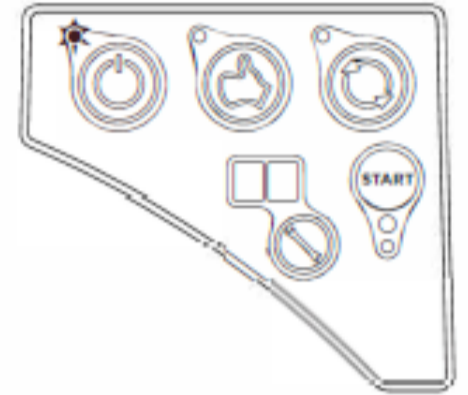


- 2 Load Machine**
PULP ONLY


Do NOT overload
4 large or 6 small
pulp products only

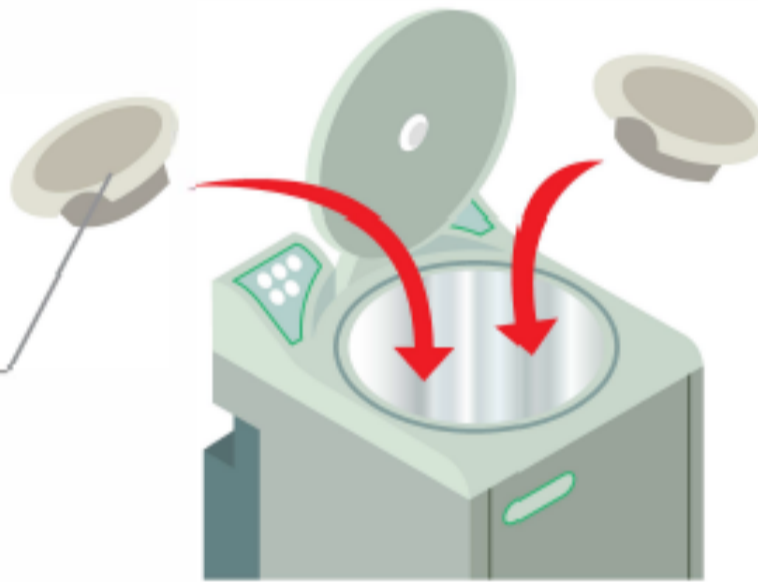


**MACHINE READY
TO START**

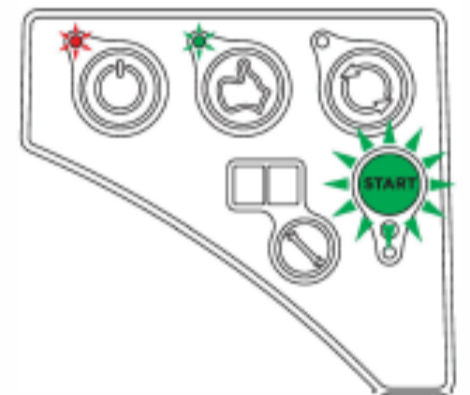

- 3 Hold hand over Green sensor to close lid & start machine**

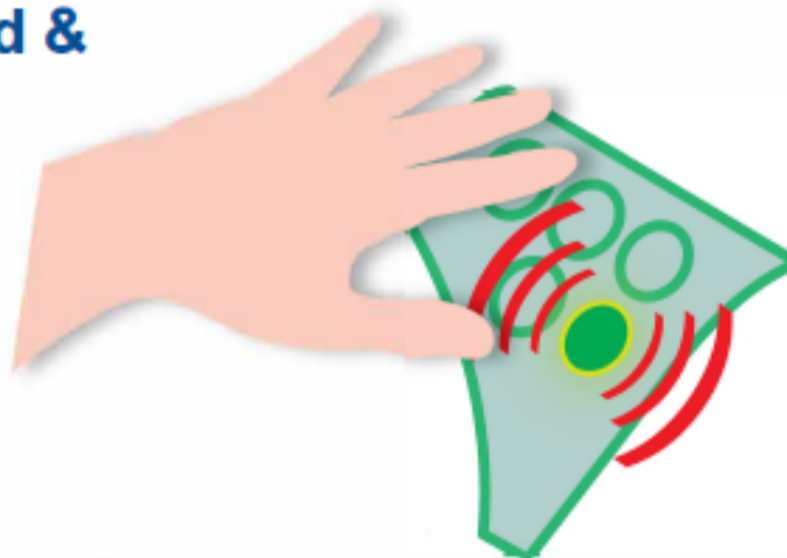

DISPOSAL CYCLE

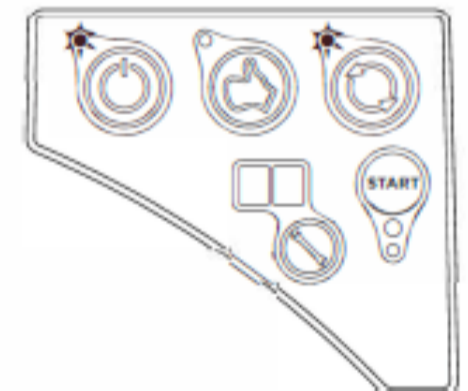

- 4 Machine finished and ready for next load. Return to step 1**

To clear most faults turn power off then back on at wall mounted isolator switch

