

# OPERATING INSTRUCTIONS


## PULPMATIC UNO

MicrobeSafe<sup>®</sup>  
Antimicrobial Surface


POS0005

### HANDS FREE OPERATION

- 1 Open Lid**  
Via optical foot sensor

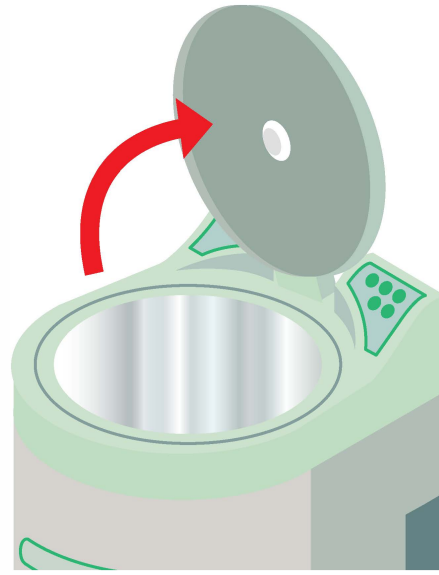


#### MACHINE READY

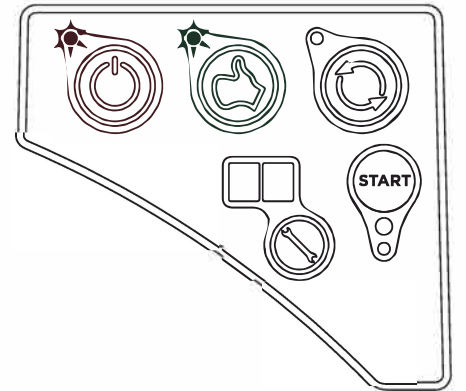


- 2 Load Machine**  
Maceratable Pulp/  
Paper/Wipes Only


**Do NOT overload**  
Single item per cycle  
pulp products only

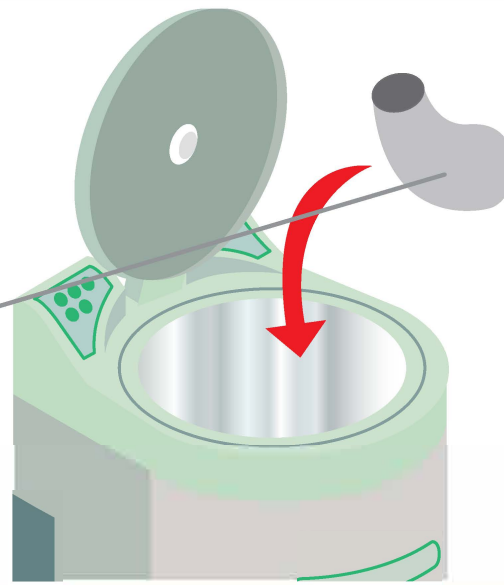


#### MACHINE READY TO START

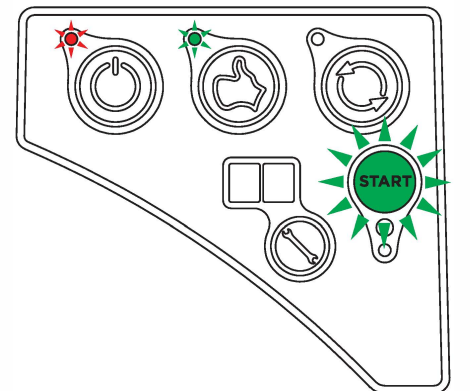

- 3 Hold hand over Green sensor to close lid & start machine**

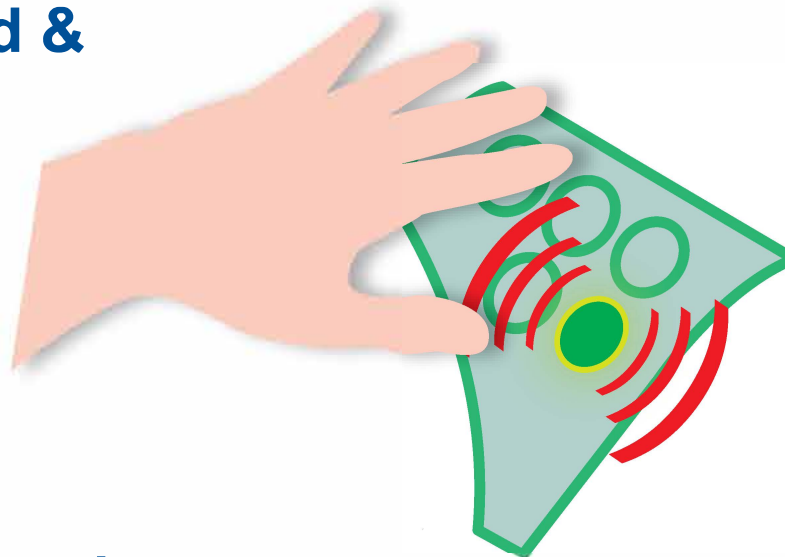

#### DISPOSAL CYCLE

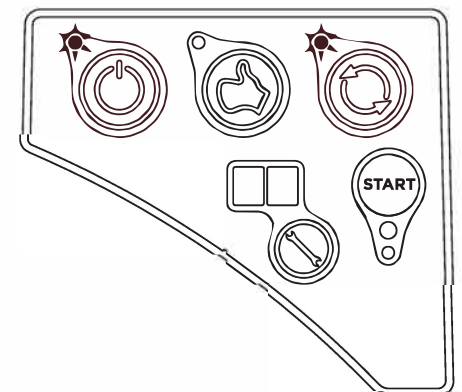

- 4 Machine finished and ready for next load. Return to step 1**

To clear most faults turn power off then back on at wall mounted isolator switch

