

Plastic Male Urine Bottle



neu INDUSTRIES
SAFER • CLEANER • BETTER



Available
in **two**
Colours



Graduation
Markings



Up to
1200ml



Reusable After
Washing



Secured
Non-drop Lid

Plastic Male Urine Bottle




It is common for people to have difficulty in bladder incontinence while recovering from an extreme operation. As such, a urine bottle is used to assist people in their sanitation needs.

The plastic urine bottle is made of reusable thermoplastic with an attached non-drop lid to assist patients with limited mobility to go about their toileting in bed when it is inconvenient to go toilet.

Compact, Light Weight

The design of the male urine bottle is hygienically safe and easy to use. It is made of thermoplastic and weighs only 140g, ideal for male users confined to bed or inconvenient to go toilet.

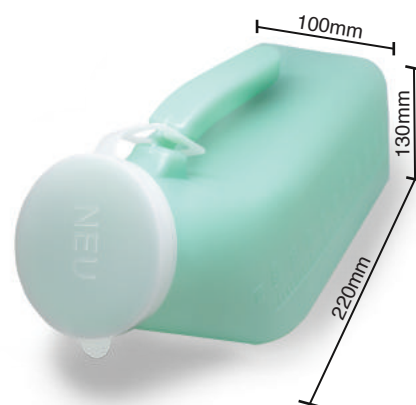

Embossed Graduation Markings

The urine bottle comes with embossed graduation markings (20ml each mark), allowing users to measure the amount of urine collected.

Tight Non-Drop Plastic Lid

The urine bottle comes with an attached non-drop lid to prevent spillage of urine. Users are advised to empty their urine in the toilet bowl after each use.



Wide 60mm Diameter Opening

The wide opening of the urine bottle is easy for cleaning and drying after use.


Guide to usage




1. Open the lid of the urine bottle before using



2. Hold the urine bottle between the legs. Urine into the bottle.

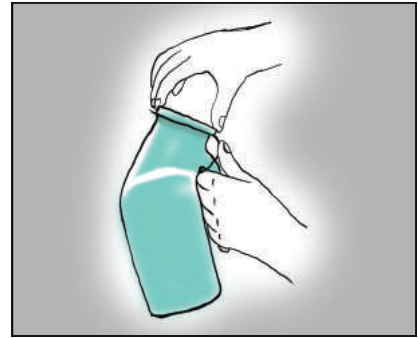

3. Gently close the lid after use. Keep the urine bottle upright.

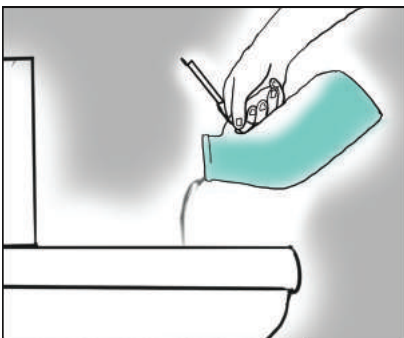

4. Empty the urine directly into the toilet bowl and flush after use.



5. Wash the urine bottle with disinfectant and rinse thoroughly with water.



6. Dry the urine bottle on a rack. Ensure bottle is dry before subsequent use.

***This product information does not provide professional medical advices**

The user guide of this product is provided for general information purposes only. It is not intended to replace consultation with a healthcare professional or to provide medical advice, diagnosis or treatment. Always seek the advice of a physician or other qualified health care provider with any medical or health questions. Never disregard professional medical advice or delay in seeking it because of something you have read about this product.

PRODUCT INFORMATION	CAPACITY	PRODUCT CODE
 <p>Plastic Male Urine Bottle with attached lid, translucent Green</p>	1200ml	PUM1000G
 <p>Plastic Male Urine Bottle with attached lid, translucent White</p>	1200ml	PUM1000W

****Disclaimer**

The user guide on this product is provided "as is" without warranties or conditions of any kind either expressed or implied. To the fullest extent possible under applicable law, NEU Industries Pte Ltd disclaims all warranties and conditions, expressed or implied, including but not limited to, implied warranties or conditions of merchantability and fitness for a particular purpose, non-infringement or other violation of rights. NEU Industries Pte Ltd does not warrant or make any other representations regarding the use, accuracy, timelines, applicability, performance, security, availability or reliability of this product, or the results from the use of this product.



132 Joo Seng Road #04-01 Singapore 368358

Tel: 6817 8969 Fax: 6636 9813

sales@neuindustries.com

www.neuindustries.com