

NEW

Pulpmatic® **FLEX**

Smarter. Safer. Healthcare.



63% Less Water*



50% Faster Cycles*



73% Less Energy*

ClearSense®
Patented intelligent monitoring system





63% Less Water*



50% Faster Cycles*



73% Less Energy*

Pulpmatic® Flex with ClearSense®

Smarter and More Efficient

The **Pulpmatic® Flex** from **DDC** delivers intelligent performance and precise engineering for healthcare settings where hygiene and reliability are vital. It also helps reduce energy, water and operating costs.

Innovative and Intelligent

ClearSense® by **DDC†**, an exclusive feature of the **Pulpmatic® Flex**, is a patented intelligent monitoring system. It detects when pulp is fully processed, allowing the machine to use the optimal amount of electricity and water to complete the cycle. This results in **measurable savings**, whilst protecting the drain network.

Exceptional Efficiency

With sustainable performance, the **Pulpmatic® Flex** runs up to **50% faster** and uses **63% less water** and **73% less energy** than other macerators*, helping healthcare providers save resources, reduce costs, and **meet Net Zero targets**.


Safer Healthcare

Built for infection control, the **Pulpmatic® Flex** features contactless operation, chemical disinfection system and antimicrobial surfaces proven to **reduce bacterial growth up to 99.99%** – providing **safer care for your staff and patients**.

*Compared with current DDC models and other brands.

†ClearSense® is available on selected models.

What is **Clear Sense**

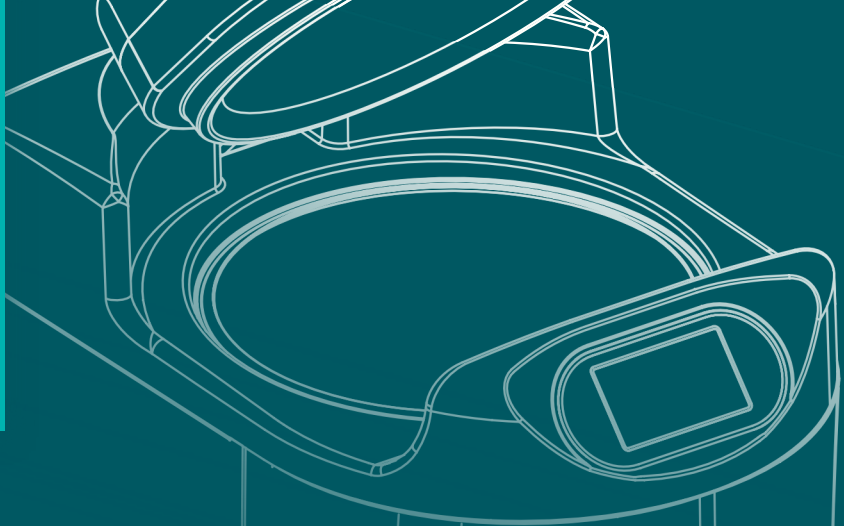

ClearSense® technology is the latest innovation from **DDC**, launched in the **Pulpmatic® Flex** macerator.

This intelligent system **optimises performance** by monitoring when the chamber is free of pulp, ensuring every cycle uses the optimum water and energy needed.


By combining **reliability** with efficiency, ClearSense® reduces operating costs and shortens cycle times leading to a cleaner, faster, and more **sustainable** maceration performance.



1

Load and Start

The user loads the pulp product into the Flex chamber and the cycle starts automatically.

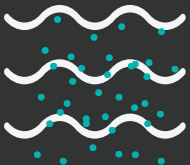

2

PreMaceration Soak

Water is injected into the chamber, softening the pulp to prepare it for maceration.



3

Maceration

High-speed blades break down the pulp which is gently discharged into the drainage system.



4

Active Monitoring & Clear Detection

When clear, the system senses and finishes automatically to save water, energy, and time.




5

Drain & Ready

Final rinse leaves the chamber and pipework clean and ready for next use.

Pulpmatic[®] Flex Key Benefits

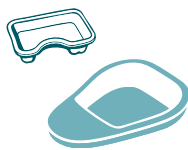

High-Capacity Performance

Processes up to six items per cycle, keeping workflows fast and efficient in busy clinical areas.



Advanced Blockage Prevention

Dual chambers stop non-maceratable items reaching the drain and help prevent costly disruptions.



Seamless, Contactless Operation

Hands-free loading with automatic lid closure ensures safe, simple use while reducing the risk of misloading.



Low Noise Output

Quiet operation supports calmer wards and more comfortable patient environments.



Built-In Audio Guidance*

Point-of-use instructions reinforce correct loading and operation, providing continual staff support.



Smart Self-Diagnostics

Intelligent monitoring keeps performance optimised and helps prevent unexpected downtime.



PAS29:1999 Certified Pulp Compatibility

Reliable maceration with full flexibility to choose your preferred certified pulp suppliers.



Intuitive 5" Interface

Clear on-screen prompts and status updates make every cycle straightforward.

Leading the way in blockage prevention and uptime.

The DDC Pulpmatic® Flex is engineered to minimise pulp residue within drainage pipes, ensuring consistent, efficient water usage.

This helps maintain peak operational performance and lowers the risk of unexpected downtime.

Its advanced dual-chamber design ensures that only fully macerated particles with an average size of 3–5mm exit the system*. By preventing larger debris and foreign objects from entering the drains, it significantly reduces blockage risks and protects pipework.

Its maceration cycle includes exclusive DDC innovations that prevent blockages and make any required maintenance quick and straightforward.



Open Flow System

The Pulpmatic® Flex protects drainage systems while delivering consistent performance. Other brands of macerators can release wastewater in sudden bursts, which can strain pipework and increase the risk of blockages.

The Flex avoids this by using a continuous open-flow process. Wastewater moves steadily through the system rather than being forced out all at once. This gentler flow helps avoid blockages and supports a healthier drainage network.

Pulpmatic® Flex




Continuous water flow from beginning until end of cycle



Waste pipe free flowing

Other brands

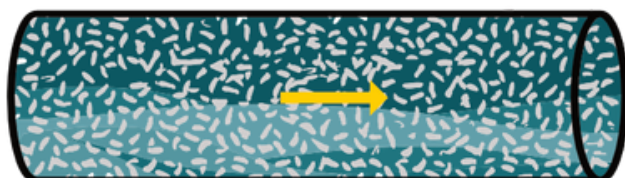

Discharge at end of cycle



Waste pipe completely full

*Results confirmed from inhouse testing.

Technical Specifications

General:

Dimensions: 450 × 605 x 1035 mm

Weight: 70 kg Dry[†]

Capacity: Up to 6* pieces of medical pulp

Operation Cycle Time: From 55 seconds

Recommended Chemical: ECOCleanse Ultimate

Chemical Dosing per Cycle: From 4ml

Decibel Rating: <70dba

Electrical:

Power Consumption Per Cycle: From 0.007 kWh

Voltage: 230V / 50 HZ

Rated Power: 0.8 kW

Motor Power: 0.75 kW

Water Pump Power: 0.21 kW

Water:

Water Consumption Per Cycle: From 10 L

Waste Height: 195mm – 235mm

Waste Connection: 50mm ID (2")

Water Pressure: 1-6 Bar

Water Tank Capacity: 17 L



Sustainability in Action

Designed for sustainability, the Pulpmatic® Flex reduces energy and water consumption while eliminating reusable cleaning cycles for cleaner, faster, and more efficient operation.

Other Models



Pulpmatic® Ultima

9-blade macerator,
High Capacity & Ideal
for General Wards



Pulpmatic® Uno

7-blade macerator,
Quiet Operation &
Ideal for Isolation
Wards



00026
Certificate Number 1161
9001/14001



NEU Industries Pte Ltd

132 Joo Seng Road, #04-01, Singapore 368358
www.neuindustries.com | sales@neuindustries.com
Tel: +65 6817 8969 | Fax: +65 6636 9813